beurer SL 15 DreamSound

DE Einschlafhilfe mit Tönen Gebrauchsanweisung.....	2
EN Sound therapy sleep aid Instructions for use.....	17
FR Aide au sommeil sonore Mode d'emploi.....	32
ES Aparato con sonido para conciliar el sueño Instrucciones de uso.....	47
IT Addormentarsi dolcemente con il suono Istruzioni per l'uso.....	62
TR Sesli uykuya dalma yardımcısı Kullanım kılavuzu.....	77
RU Помощь при засыпании со звуком Инструкция по применению.....	92
PL Urządzenie muzyczne ułatwiające zasypianie Instrukcja obsługi.....	106

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Read these instructions for use carefully and keep them for later use. Make sure they are accessible to other users and take account of the information they contain.

Dear customer,

Thank you for choosing a product from our range. Our name is synonymous with high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,
Your Beurer team

Contents

1. Getting to know your device	19
2. Included in delivery.....	20
3. Signs and symbols	20
4. Intended use.....	21
5. Warnings and safety notes.....	22
6. Device description.....	24
7. Initial use	25
8. Usage	26
9. Cleaning and maintenance.....	28
10. Troubleshooting.....	28
11. Disposal	29
12. Technical specifications	29
13. Warranty/service.....	30

1. Getting to know your device

Why is sleep so important?

A restorative night's sleep is of great importance for our physical and psychological well-being; after all, it is during sleep that we process new experiences and improve our mental and physical fitness, among other tasks.

The beurer SL 15 DreamSound sound therapy sleep aid has been specially design for making it easier to get to sleep. This sleep aid offers four calming melodies for relaxation – ideal preparation for helping you fall asleep.

Why is falling asleep so difficult for us?

When we encounter stress and anxiety, the autonomic nervous system puts our bodies into a state of alarm. The body then excretes the stress hormone cortisol and inhibits the production of the sleep hormone melatonin, resulting in sleeplessness. This sleep aid helps you actively return your autonomic nervous system to a state of balance. Its calming sounds distract you from your thoughts and slow down your brain activity. The release of cortisol declines while the production of the happiness hormone dopamine rises, enabling you to fall asleep more quickly and easily.

2. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that the device and accessories have no visible damage and all packaging material is removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1x sleep aid
- 3x AAA batteries

3. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	Read the instructions for use	 Note	Important information to note
 WARNING	Warning notice indicating a risk of injury or damage to health	 IMPORTANT	Safety note indicating possible damage to the device/accessories
	Manufacturer	 PAP	Dispose of packaging in an environmentally friendly manner

	<p>This product satisfies the requirements of the applicable European and national directives.</p>		<p>Disposal in accordance with the EC Directive on Waste Electrical and Electronic Equipment (WEEE).</p>
	<p>Certification symbol for products that are exported to the Russian Federation and CIS countries</p>		<p>Do not dispose of batteries containing harmful substances with household waste</p>

4. Intended use

WARNING

This sleep aid is for use by humans only. Do not use the sleep aid on animals. This sleep aid is only intended for use indoors and in a domestic environment. This sleep aid is not intended for medical or commercial use. This sleep aid is not intended for therapeutic use to treat sleep disorders that result from health problems. Use the sleep aid only for the purpose for which it was developed, and in the manner specified in these instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use.

5. Warnings and safety notes

WARNING

- Keep this sleep aid and associated packaging material away from children. Choking hazard!
- This device may be used by people with reduced physical, sensory or mental abilities or a lack of experience or knowledge, provided that they are supervised or have been instructed in how to use the device safely, and understand the resulting risks.
- Children must not play with the device.
- Children must not perform cleaning and user maintenance without supervision.
- Protect the device from knocks, damp, dust, chemicals, significant fluctuations in temperature, direct sunlight, splashes, and sand, and ensure that sources of heat are at a sufficient distance (ovens, heaters).
- Never immerse the device in water or other liquids.

General notes on handling batteries

WARNING


- If your skin or eyes come into contact with battery fluid, wash out the affected area with water and seek medical assistance.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- Risk of explosion! Do not throw batteries into a fire.
- Choking hazard! Small children may swallow and choke on batteries. Therefore, store batteries out of reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- Do not charge or short-circuit the batteries.

- If the device is not to be used for a long period of time, remove the batteries from the battery compartment.
- Only use an identical or equivalent battery type.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

 **IMPORTANT**

- Do not open the device. Do not attempt to repair the device yourself. This could result in serious injury. Failure to comply with this instruction will void the warranty.
- For repairs, please contact Customer Services or an authorised retailer.
- Do not drop the device. Never throw the device. Protect the device against impact and powerful forces.
- Use and store the sleep aid at an ambient temperature between 10 °C and 40 °C. Higher or lower temperatures may damage the sleep aid.

6. Device description

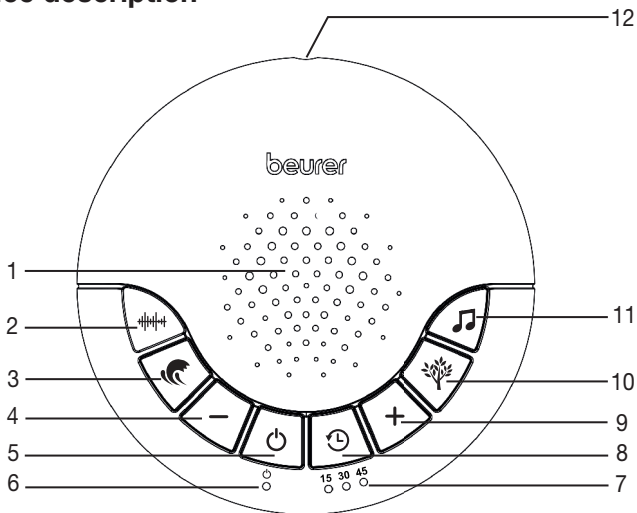
1 Speaker	7 Timer display (15, 30 or 45 minutes)
2 Melody button 1: hair dryer	8 Automatic switch-off button
3 Melody button 2: sounds of the sea	9 Volume increase button (+)
4 Volume decrease button (-)	10 Melody button 3: summer's night
5 ON/OFF button	11 Melody button 4: harmony
6 Readiness indicator	12 AUX connection

7. Initial use

Inserting the batteries

Before initial use of the sleep aid, you first need to insert the three AAA batteries including in delivery into the device. To do so, proceed as follows:
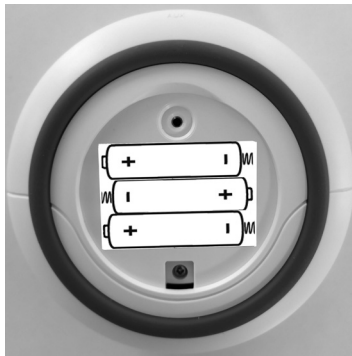
1. The battery compartment cover is located on the underside of the sleep aid. Use a cross-head screwdriver to unscrew the battery compartment cover.
2. Remove the battery compartment cover.
3. Place the three AAA batteries included in delivery into the battery compartment as shown. When inserting the batteries, ensure that the correct polarity (+/-) is observed.
4. Use a cross-head screwdriver to screw the battery compartment cover back in place. Make sure that the battery compartment is securely closed.

8. Usage


1. Position the sleep aid within arm's reach (e.g. on your bedside table).
2. Press the ON/OFF button  to switch the sleep aid on. The operating display will light up in blue.
3. The sleep aid has 4 different melodies. Select one of the melodies stated below using the respective button (hair dryer, sounds of the sea, summer's night, or harmony).

	Hair dryer		Summer's night
	Sounds of the sea		Harmony

4. Use the **+**/**-** buttons to select your preferred volume.
5. The device features a timer with automatic switch-off. After the timer period has elapsed (15, 30, or 45 minutes), the sleep aid will switch off automatically. Select your preferred session time (15, 30, or 45 minutes) using the automatic switch-off button. The selected session time lights up in blue on the sleep aid.
6. Press the ON/OFF button  to switch the sleep aid off. The operating display turns off.

AUX connection

You can also listen to the various melodies via headphones or an external speaker – just connect your headphones/speaker to the AUX connection on the sleep aid using a jack (3.5 mm).

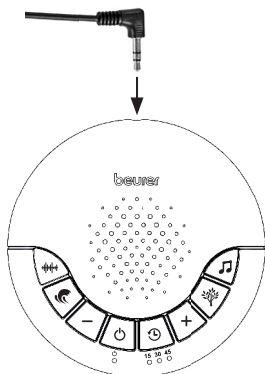i Note

It is not possible to connect headphones or a speaker to the sleep aid via *Bluetooth*®.

9. Cleaning and maintenance

WARNING

- Do NOT clean the sleep aid in the washing machine or dishwasher. Do not use any chemical or abrasive cleaning agents for cleaning.
- Clean the sleep aid with a slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution.

10. Troubleshooting

Problem	Possible cause	Solution
Device will not switch on	Flat batteries	Insert new batteries
Sounds are very difficult to hear	Batteries almost flat	Insert new batteries
The device will not switch on although new batteries have been inserted	The device is faulty	Contact Customer Services

11. Disposal

For environmental reasons, do not dispose of the device in household waste at the end of its service life.

Dispose of the device at corresponding collection points in your country. Dispose of the device in accordance with the EC Directive on Waste Electrical and Electronic Equipment (WEEE). If you have any questions, please contact the local authorities responsible for waste disposal.

The used, completely flat batteries must be disposed of via specially designated collection containers or special waste collection points, or via electronics retailers. You are legally required to dispose of the batteries.

- The codes below are printed on batteries containing harmful substances:

Pb = battery contains lead,
Cd = battery contains cadmium,
Hg = battery contains mercury.

12. Technical specifications

Type	SL 15
Item number	677.66
Weight	approx. 130 g (excluding batteries)
Battery	3 x 1.5 V --- AAA LR03 battery
Device dimensions	approx. \varnothing 12 x 3.7 cm
Battery life	approx. 36 hours

Subject to technical changes.

13. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.
German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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Beurer GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)